


INSTRUCTIONS FOR USE

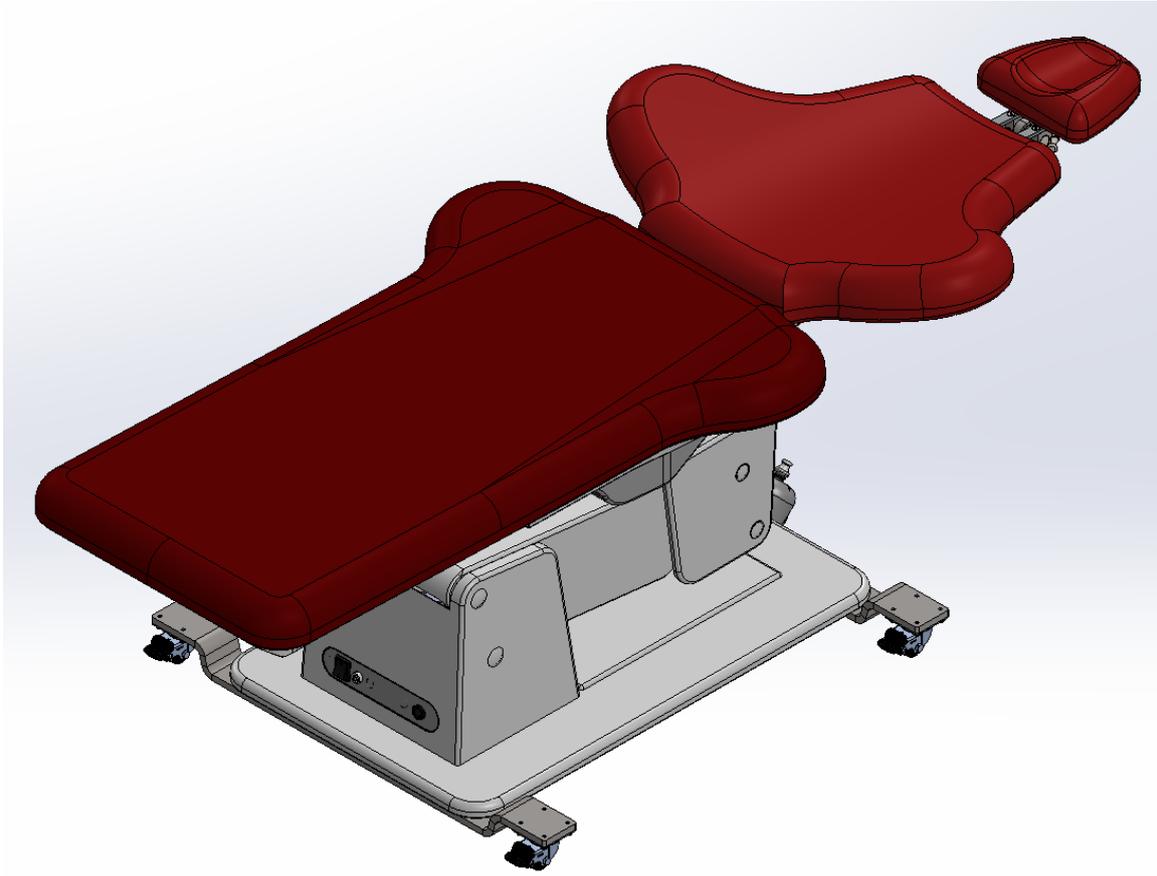


Mobile Kit Classic Accessories for UnicLine Chair and Patient Chair+

Description of the equipment - SB-1390

Mobile Kit Classic for UnicLine Chair (SB-0800) and Patient Chair+ (SB-0820) is an accessory consisting of a wheel set with four wheels, designed for mounting on the patient chair and - between dental treatments - to make the patient chair physically movable.

Overview of Mobile Kit Classic

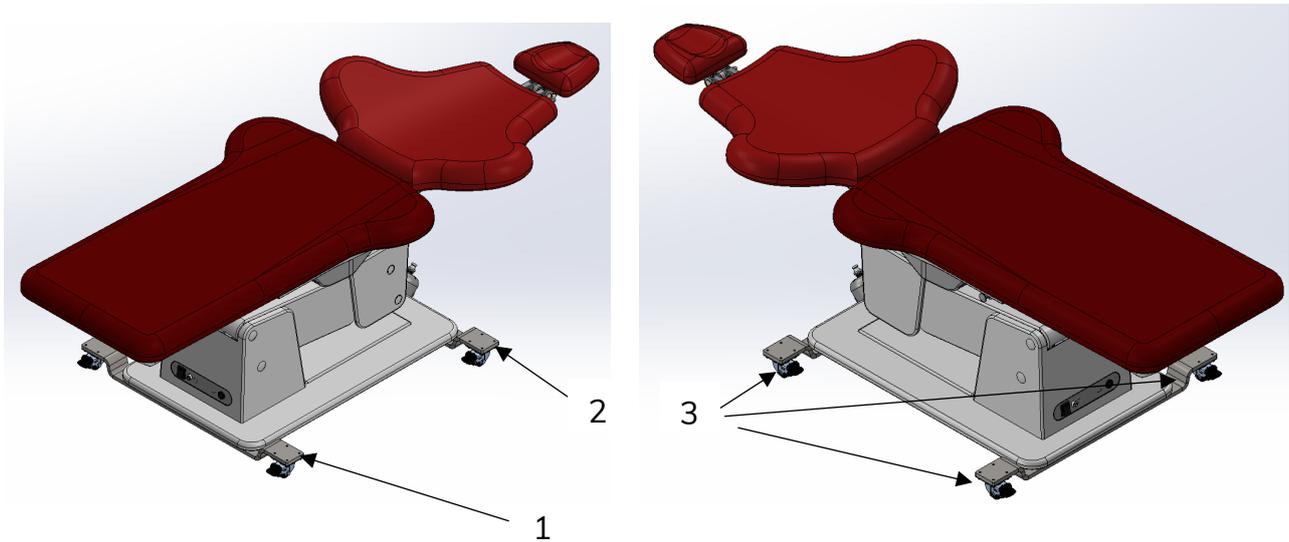

1	Wheel bracket for patient foot end
2	Wheel bracket for patient head end
3	Wheels with brakes on all four wheels

Safety instructions

The enclosed safety instructions are exclusively for the Mobile Kit Classic accessory.

Please note the general safety instructions in the UnicLine Chair (KA-0033) and Patient Chair+ (KA-0034) operating instructions.

Mobile Kit Classic may only be mounted on patient chairs that are used on floors made of concrete, ceramic tiles or other non-combustible material in accordance with the above-mentioned installation instructions (KA-0033 and KA-0034).

WARNING

Damaged cable.

Electric shock due to a damaged 230V cable can cause serious injury or death.

- Check the 230V cable for damage after the move.
- Do not use the patient chair if the 230V cable is damaged. Switch off the patient chair immediately if it is already switched on.
- Contact a Heka Certified Technician (HCt) if the 230V cable needs to be replaced.

CAUTION

Collision

Collision with patients, furniture and other equipment can cause minor to moderate injuries.

- Do not move/drive the patient chair with the Mobile kit attached with patients in it.
- There must be no patients in the room while the patient chair with the Mobile kit attached is being moved.
- Check that the area is clear of obstacles before moving the patient chair with the Mobile kit attached.
- Make sure that the patient chair with Mobile kit attached is disconnected from other equipment before disengaging the wheel brakes and moving the chair.
- Check that the wheels (all four) are locked when the chair with the Mobile kit attached is moved to the new location.
- Make sure that the new location of the chair with the Mobile kit attached has sufficient distance to walls and other obstacles, and that the distance to the dental unit is in accordance with the operating instructions.

NOTICE

Patient weight limitations

The UnicLine Chair (SB-0800) and Patient Chair+ (SB-0820) are EN/ISO 6875 approved for a load of 179 kg (398 lb).

The maximum load of 179 kg (398 lb) also applies to the Patient Chair with Mobile Kit Classic attached.

If this weight limit is exceeded, the patient chair may not function properly.

- Make sure that no patients exceeding the weight limit are placed in the patient chair.

Introduction to operation

The wheel brake is activated and deactivated with your foot.



Wheel brake activated



The wheel brake is switched off

To move the chair:

- Disconnect the chair from other equipment and 230V cable
- Switch off the wheel brake on all four wheels
- Push the chair slowly with both hands

When the chair is moved:

- Place the chair with the correct distance to the dental unit and observe the distance to walls and other obstacles.
- Activate the wheel brake on all four wheels
- Connect the chair according to the instructions

Cleaning of the chair

The Mobile kit is cleaned according to the same conditions as the recommendations for cleaning the patient chair described in the operating instructions for Patient Chair and Patient Chair +, respectively.

Annual service

NOTICE

Annual maintenance

Lack of maintenance can lead to malfunctions and result in material damage.

- The user is responsible for ensuring that the equipment undergoes annual maintenance and for continuously checking that all functions are working properly.
- Service and maintenance may only be performed by Heka Certified Technicians (HCt).
- Do not perform service or maintenance with a patient seated in the chair.

Check during annual service:

- Brake function on all four wheels.
- Unhindered swivelling and rolling movement on all four wheels.

HEKA

Heka Dental A/S
Litauen Alle 4
DK-2630 Taastrup
Denmark
T: +45 4332 0990
M: info@heka-dental.dk
heka-dental.com



Homepage



Facebook



LinkedIn



Instagram