


# INSTRUCTIONS FOR USE

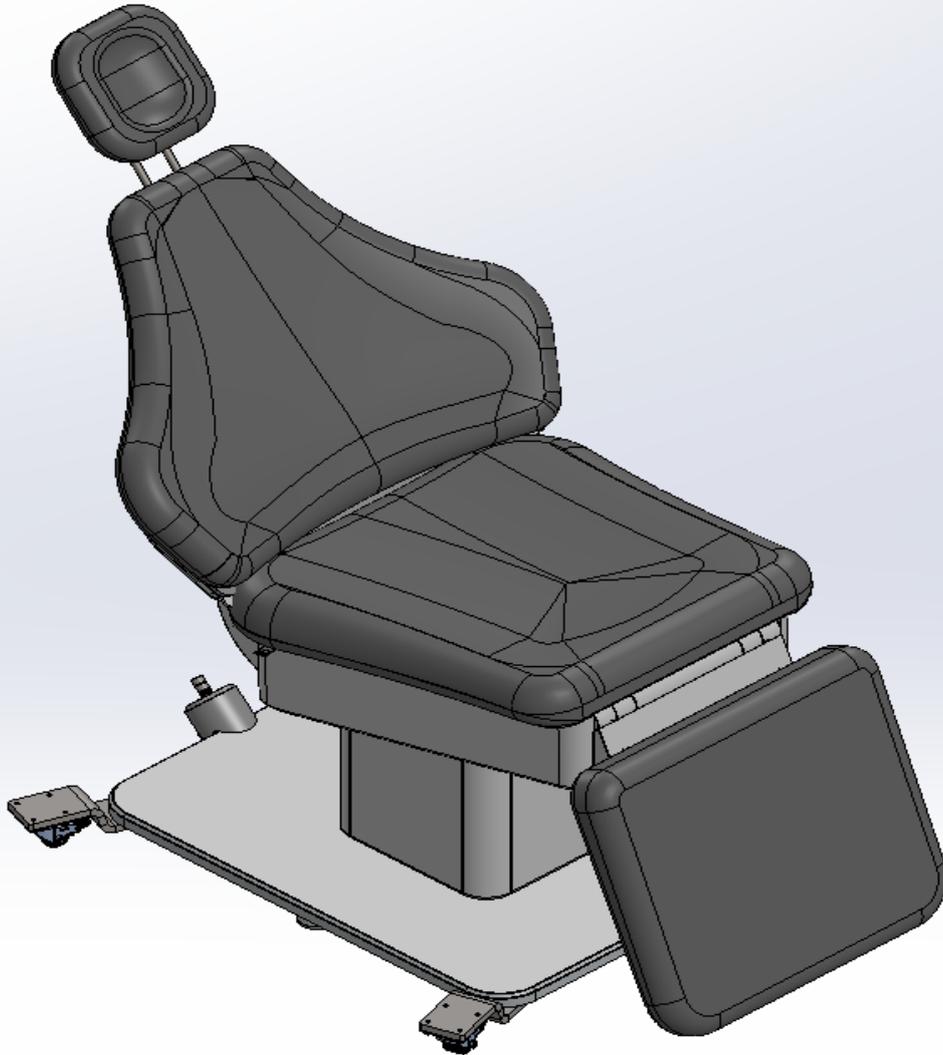


## Mobile Kit Heka Patient Chair

Accessories for  
Heka Patient Chair

## Description of the equipment - SC-50708

The Heka Patient Chair Mobile Kit is an accessory consisting of a wheel set with four wheels, designed for mounting on the patient chair and - between dental treatments - to make the patient chair physically movable.

## Overview of Mobile Kit Heka Patient Chair

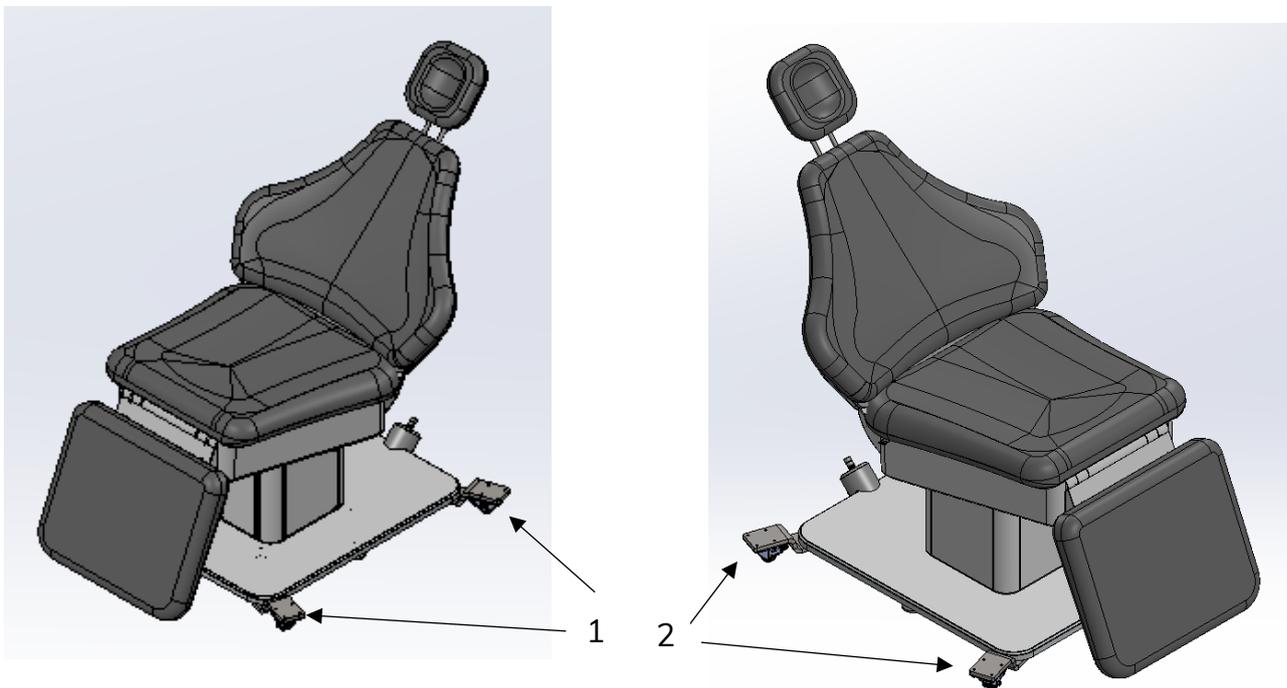

1	Wheel mounts
2	Wheels with brakes on all four wheels

## Safety instructions

The enclosed safety instructions are exclusively for the Mobile Kit Heka Patient Chair accessory.

Please note the general safety instructions in the assembly instructions (KA-2010) and instructions for use (KA-2050) for the Heka Patient Chair.

Mobile Kit Heka Patient Chair may only be mounted on patient chairs that are used on floors made of concrete, ceramic tiles or other non-combustible material in accordance with the above-mentioned installation instructions (KA-2010).

### **WARNING**

#### **Damaged cable.**

Electric shock due to a damaged 230V cable can cause serious injury or death.

- Check the 230V cable for damage after moving.
- Do not use the patient chair if the 230V cable is damaged. Switch off the patient chair immediately if it is already switched on.
- Contact a Heka Certified Technician (HCt) if the 230V cable needs replacing.

### **CAUTION**

#### **Collision**

Collision with patients, furniture and other equipment may result in minor to moderate injury.

- Do not move/drive the patient chair with the Mobile kit attached with patients in it.
- There must be no patients in the room while the patient chair with the Mobile kit attached is being moved.
- Check that the area is clear of obstacles before moving the patient chair with the Mobile kit attached.
- Make sure that the patient chair with Mobile kit attached is disconnected from other equipment before disengaging the wheel brakes and moving the chair.
- Check that the wheels (all four) are locked when the chair with the Mobile kit attached is moved to the new location.
- Make sure that the new location of the chair with the Mobilkit attached has sufficient distance to walls and other obstacles, and that the distance to the dental unit is in accordance with the operating instructions.

## **⚠ CAUTION**

### **Securing the patient chair when removing the Mobile Kit**

Failure to secure the patient chair after removing the Mobile kit may result in minor to moderate injury.

- Ensure that the patient chair is bolted to the floor as specified in the Assembly Instructions (KA-2010) if the Mobile Kit Heka Patient Chair is removed and the patient chair is no longer to be movable.

## **NOTICE**

### **Patient weight limitations**

The Heka Patient Chair is EN/ISO 6875 approved for a load of 200 kg (440.9 lb).

The maximum load of 200 kg (440.9 lb) also applies to patient chairs with the Mobile Kit Heka Patient Chair attached.

If this weight limit is exceeded, the patient chair may not function properly.

- Make sure that no patients exceeding the weight limit are placed in the patient chair.

## Introduction to operation

The wheel brake is activated and deactivated with your foot.



Wheel brake activated



The wheel brake is switched off

### To move the chair:

- Disconnect the chair from other equipment and 230V cable
- Switch off the wheel brake on all four wheels
- Push the chair slowly with both hands

### When the chair is moved:

- Place the chair with the correct distance to the dental unit and observe the distance to walls and other obstacles.
- Activate the wheel brake on all four wheels
- Connect the chair according to the instructions

### Cleaning of the chair

The mobile kit is cleaned according to the same conditions as the recommendations for cleaning the patient chair described in the Heka Patient Chair user manual (KA-2050).

## Annual service

### NOTICE

#### Annual maintenance

Lack of maintenance can lead to malfunctions and result in material damage.

- The user is responsible for ensuring that the equipment undergoes annual maintenance and for continuously checking that all functions are working properly.
- Service and maintenance may only be performed by Heka Certified Technicians (HCt).
- Do not perform service or maintenance with a patient seated in the chair.

Check during annual service:

- Brake function on all four wheels.
- Unhindered swivelling and rolling movement on all four wheels.

# HEKA

Heka Dental A/S  
Litauen Alle 4  
DK-2630 Taastrup  
Denmark  
T: +45 4332 0990  
M: info@heka-dental.dk  
heka-dental.com



Homepage



Facebook



LinkedIn



Instagram